


LTV[®] 1200 ventilator emergency set-up card


For 24-hour assistance, call 800.754.1914



Step 1: Choose a power source.


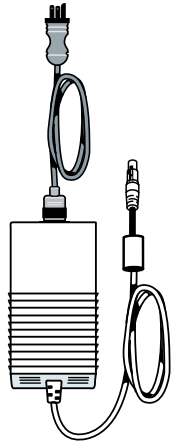


AC adapter
(Use AC power when available).

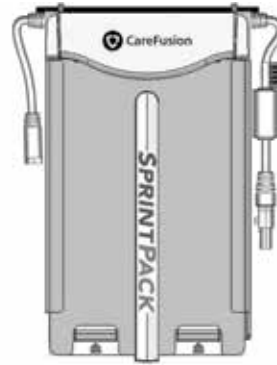


SprintPack Lithium-Ion
Power System

Step 2: Connect the LTV® 1200 ventilator to the power source.

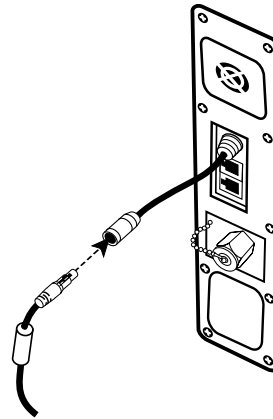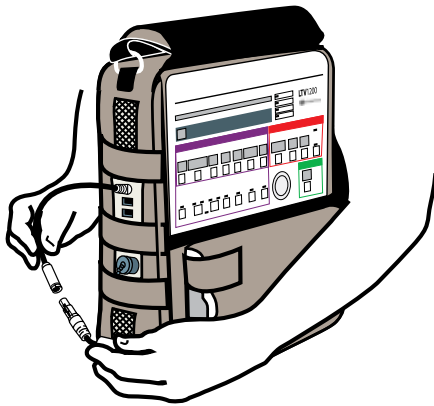

Step 3: Choose a circuit.

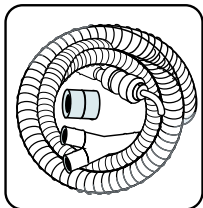

- Open bag.
- Remove patient circuit and **22 mm adapter** from bag.

Humidifier circuit (optional)




22 mm patient circuit, adult

>20 kg

>44 lb

22 mm adapter

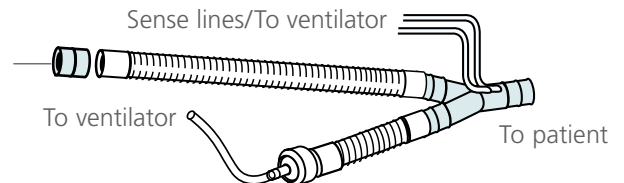


15 mm patient circuit, infant/pediatric

5 kg to ≤ 20 kg


11 lb ≤ 44 lb

22 mm adapter

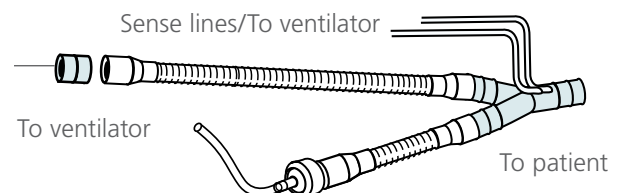


For 24-hour assistance, call 800.754.1914


Step 4: Connect the circuit to the LTV 1200 ventilator.

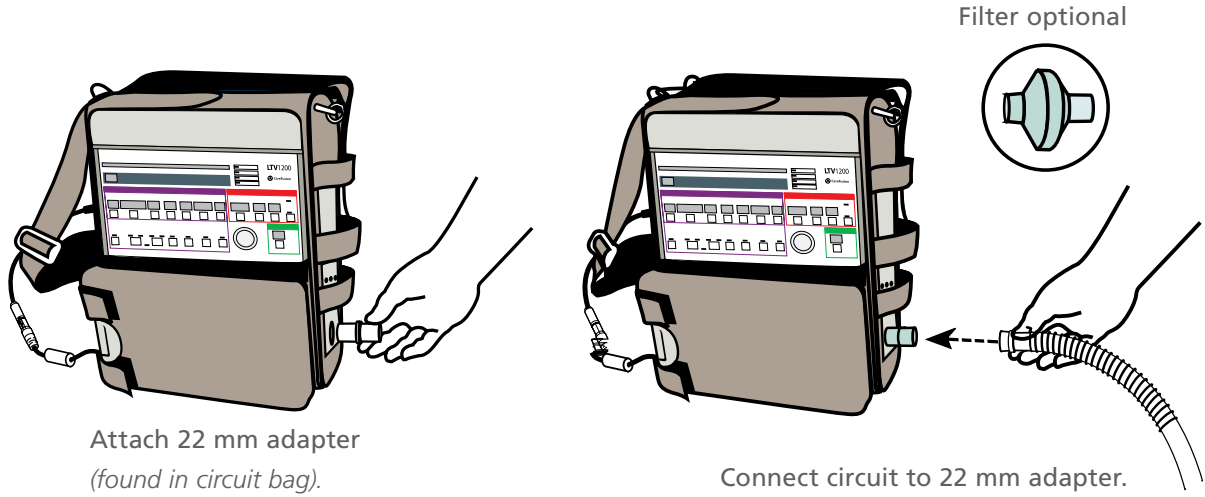


Step 5: Connect the sense lines to the LTV 1200 ventilator.

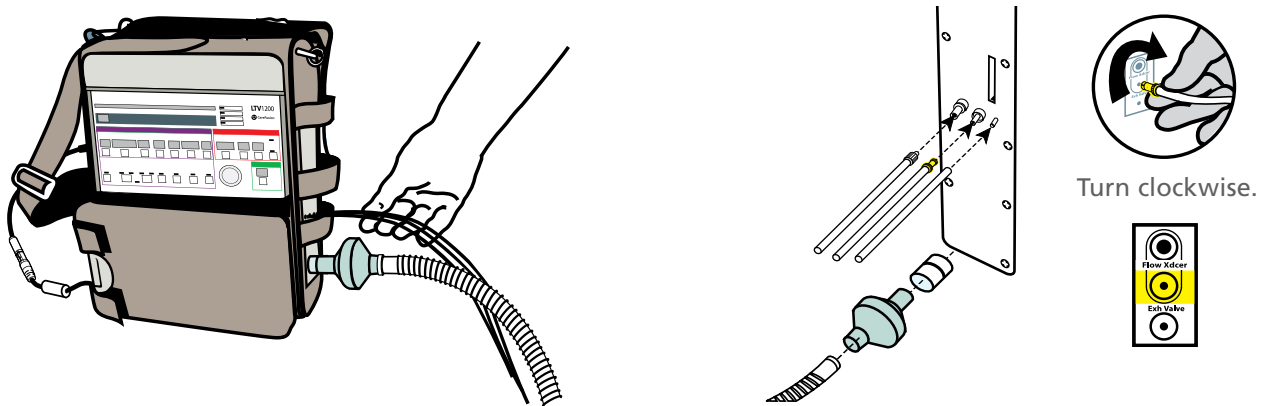

Step 6: Turn on the LTV 1200 ventilator.

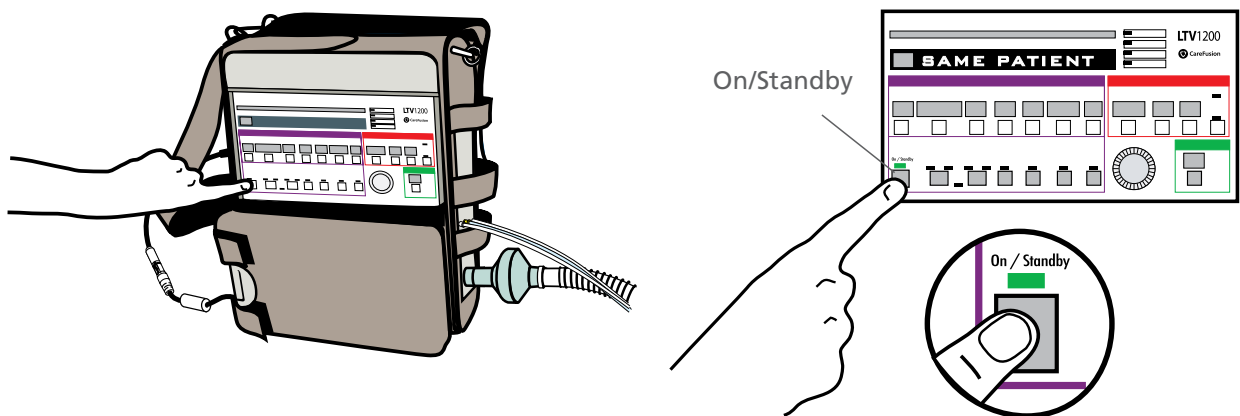


Step 7: Patient setting displays.

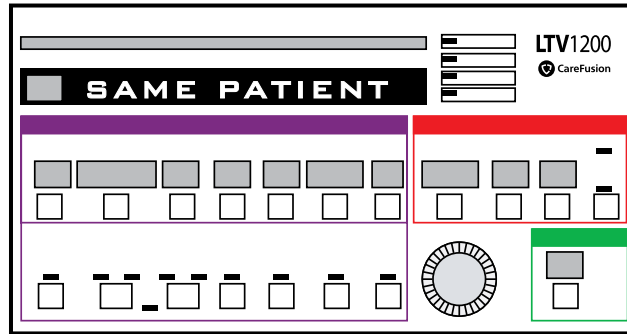


Once the power is on, see Same Patient display.

Step 8: Change display to the New Patient setting.

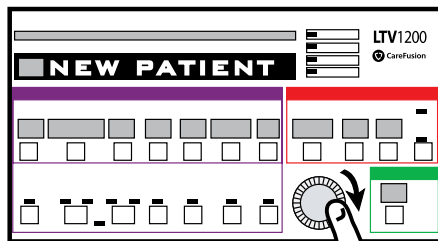


Rotate **Set Value** knob clockwise until **New Patient** is displayed.

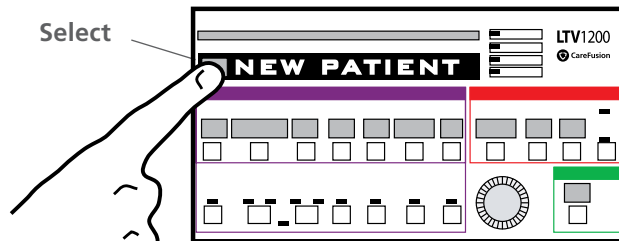


Press **Select** when **New Patient** is displayed.

Step 9: Set patient size.

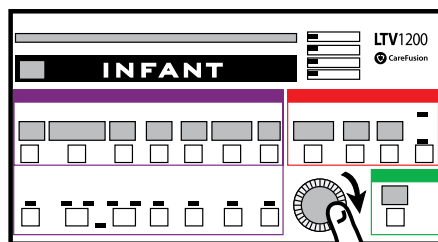


Rotate **Set Value** knob to correct patient size (*infant, pediatric or adult*).

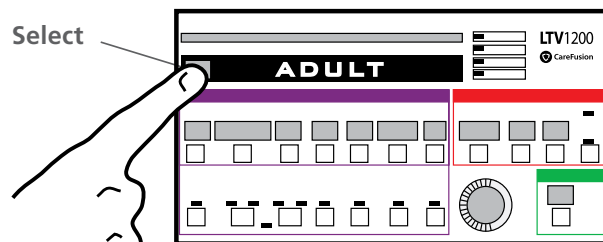

Press **Select** when proper patient size is displayed.


Note: If you make a mistake, turn the LTV 1200 ventilator off by holding the On/Standby button for three seconds. Then return to Step 6.

Step 10: Connect circuit to ET tube or mask.

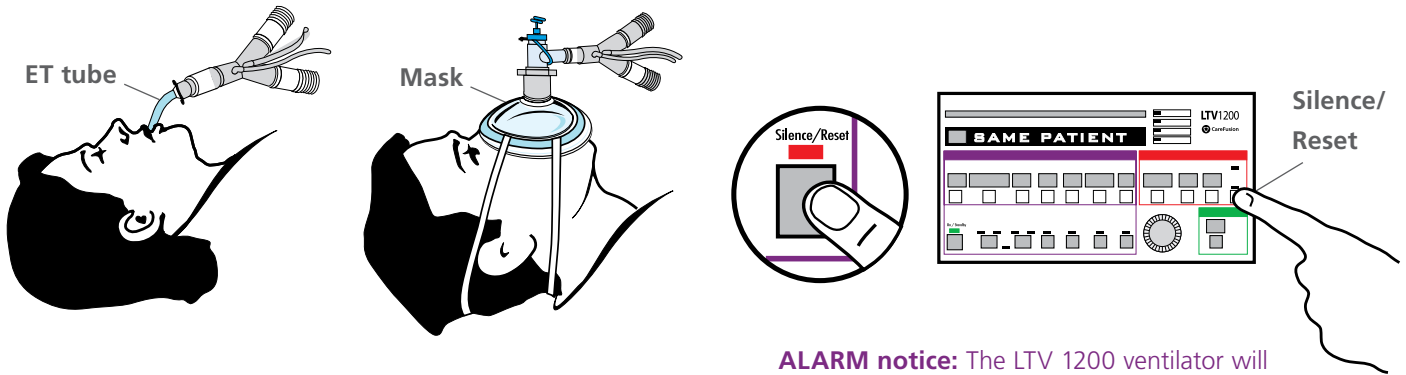


ALARM notice: The LTV 1200 ventilator will alarm until it is connected to the patient. To turn it off, you may press the **Silence/Reset** button.

Step 11: Optional: Use a mask.

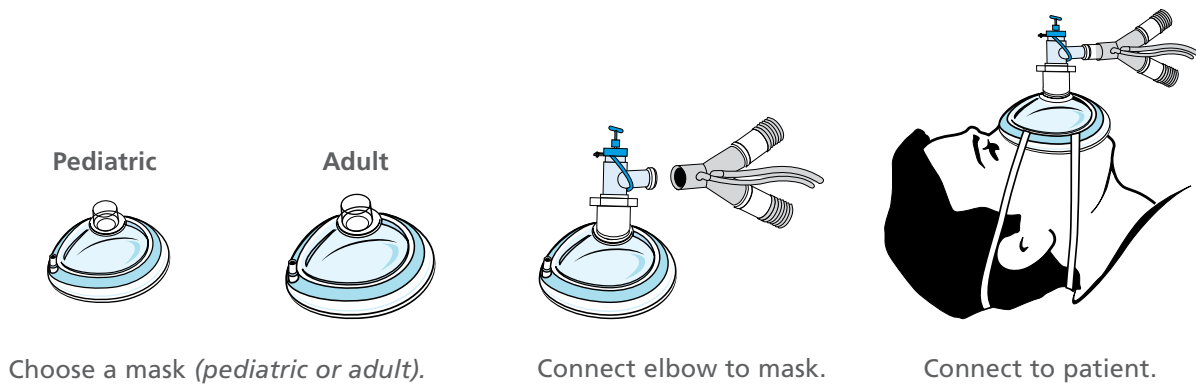

Step 12: Optional: Use an HME and/or flextube as shown for patient comfort.

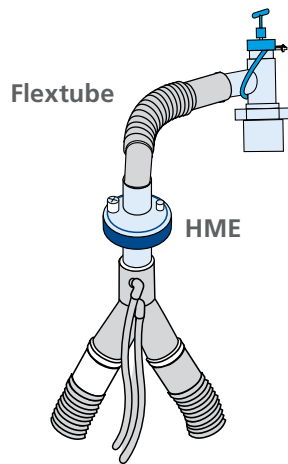

This emergency set-up card is intended to provide emergency instructions for initial LTV 1200 ventilator set-up. The initial settings may need to be adjusted based on the patient's clinical condition. For complete information regarding set-up and operation, refer to the LTV 1200 ventilator operator's manual.



CareFusion
22745 Savi Ranch Parkway
Yorba Linda, CA 92887

800.231.2466 toll-free
714.283.2228 tel
714.283.8493 fax



carefusion.com



© 2013 CareFusion Corporation or one of its subsidiaries. All rights reserved. LTV, CareFusion and the CareFusion logo are trademarks or registered trademarks of CareFusion Corporation or one of its subsidiaries. RC2310 (0813/500)